

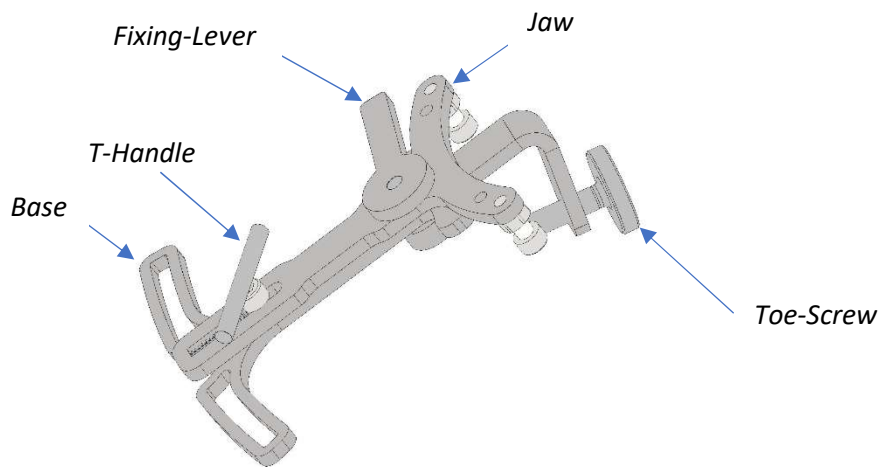


The Nail-it consists of the following parts:

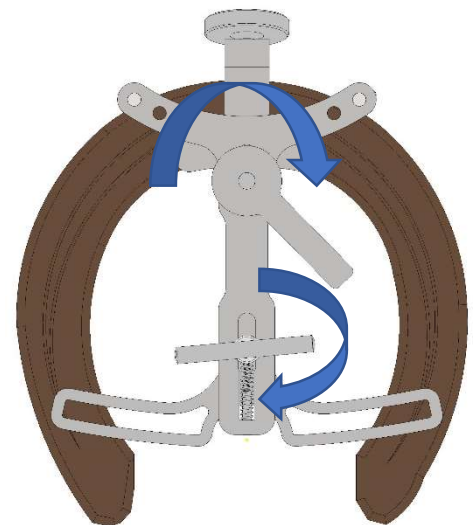


Instructions for use of the HSF Nail-it.

- Release T-Handle half a turn.
- Position the Fixing Lever at 3 o'clock.
- Hold the shoe ground surface towards you.
- Place the Nail-It on top holding it at the wide part of the base.
- Press end of the Jaw with the thumb and release to clamp it.

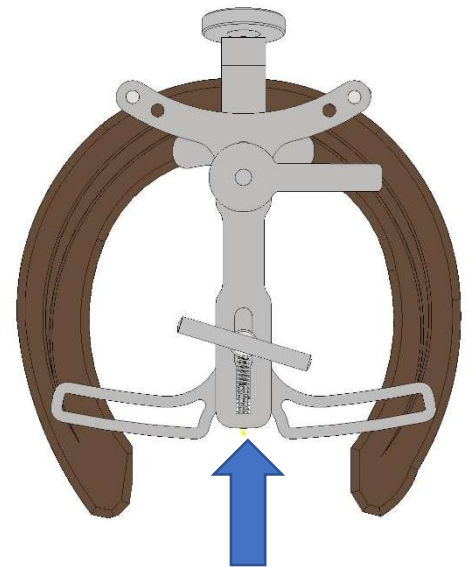


- Centre the Nail-It.
- Tighten the T-Handle followed by the Fixing Lever.

- Place the horseshoe on the hoof.
- Adjust the Toe-Screw to the obtain the desired breakover.

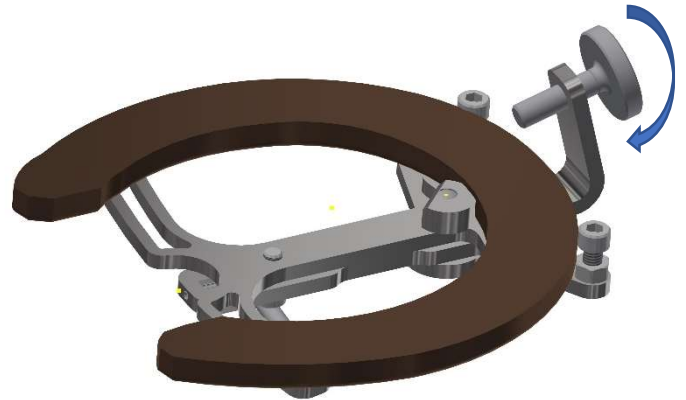

- Secure horseshoe by driving in one nail on both sides.
- Release Fixing Lever (10 o'clock)
- Remove the Nail-It.
- Finish nailing.

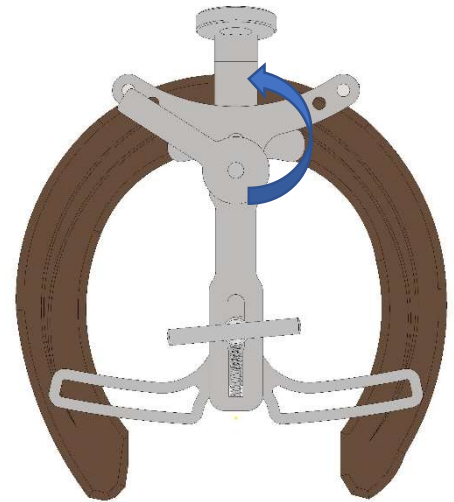

The Jaw contains two extra screw holes in order to adjust the clamp for smaller range of shoes.